

People First

People First is our promise to empower people to communicate freely, interact naturally and participate actively

INSTRUCTIONS FOR USE

MICROPHONE

105843US/12.10



What's in the box?

Microphone



Carrying pouch



Lanyard



Power supply (charger)



Important Information





This manual




Product overview


- Status indicator (orange)
- Microphone input (Voice)
- Power indicator (green)
- On/Off button
- Microphone input (noise detector), do not cover

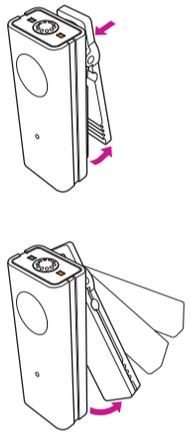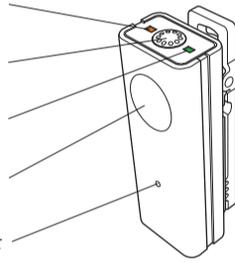


How the Microphone works

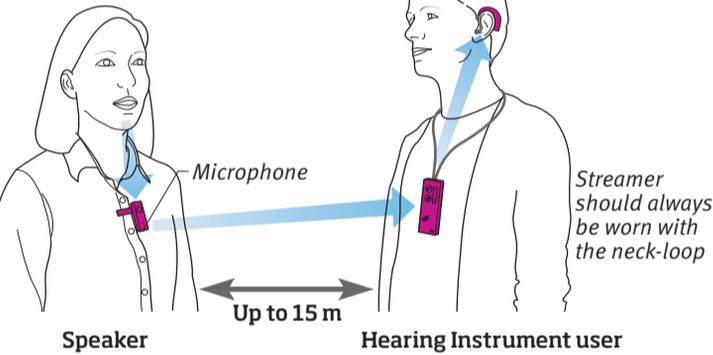


Wearing with Clip

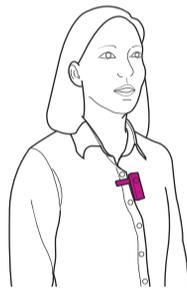

Wearing with Lanyard

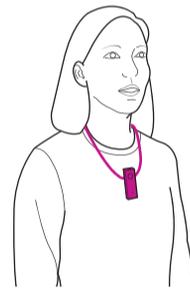



Incorrect Placement

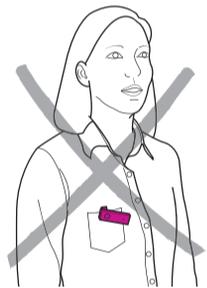

The Microphone will transmit the speech from the Speaker directly to the Hearing instrument user.
 Note: A Microphone will only work with one Streamer at a time

The microphone should point towards the speaker's mouth within a distance of 10 to 20 cm.

Charging/power indicator

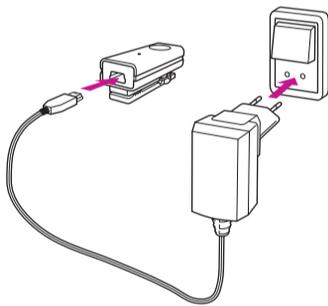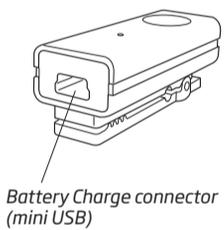

Constant red light



Battery Low
 Maximum 20 min. speaking time left

Blinking green light



Charging


Constant green light




Fully charged
 (Approx. 1.5 hours charge time.) Will last for 5 approx. hours use.

Pairing/initial setup (only needed once)

1. Streamer in pairing mode

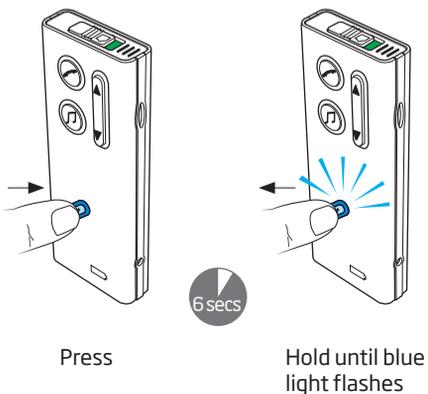

2. Microphone in pairing mode

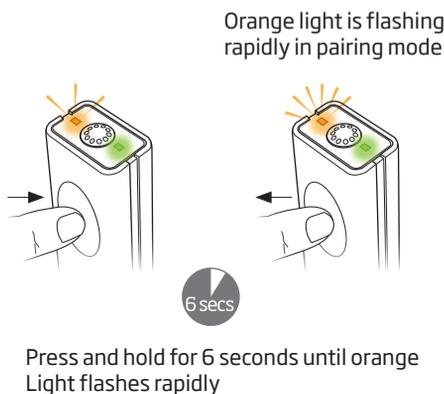

3. Microphone paired



People First

People First is our promise to empower people to communicate freely, interact naturally and participate actively

INSTRUCTIONS FOR USE

MICROPHONE




Start/turn on/resume sound from Microphone

1. Make sure Streamer is on

2. Start /turn on Microphone

3. Sound from Microphone to Streamer

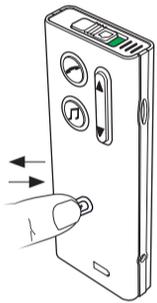

Short press on connect button




Blue light slowly pulsating on Streamer



Short press on on/off



Orange light slowly pulsating

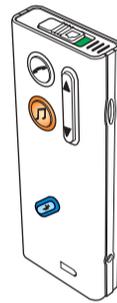

Orange light Streamer on and orange light Microphone on

Pause sound

a. Pause sound from Microphone

b. Pause sound from Streamer



Short press on on/off button



Orange light on Microphone turns off



Orange light on Streamer turns off



Short press on Streamer audio button



Orange light Streamer off



Orange light Microphone turns off

Resume sound from Streamer

1. Microphone has to be on


2. Start sound from Streamer

3. Sound from Microphone to Streamer




(green light on)

Two short presses
1 → 2 →

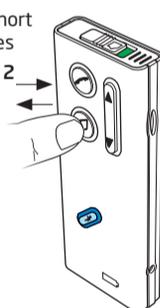

two short presses on audio button

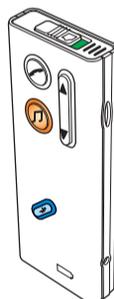

Orange light Streamer slowly pulsating



After 2 seconds, orange light Microphone turns on

Check sound

Turn off Microphone



Tap the top of the microphone to check that the sound is heard in the hearing instruments



Press and hold two seconds, until green light turns off