

People First

People First is our promise to empower people to communicate freely, interact naturally and participate actively

GETTING STARTED

Streamer Pro

128148US/12.12

Please read this user guide before you start using your new Streamer



Contents of the Streamer Pro box

The Streamer Pro box contains the following items:



Your new Streamer Pro



Cleaning cloth



Neck loop - medium length



Neck loop - long length



Charger



*Optional


Box 3

Charge base (see page 22 in IFU)



Box 4

Protective skin (see page 62 in IFU)

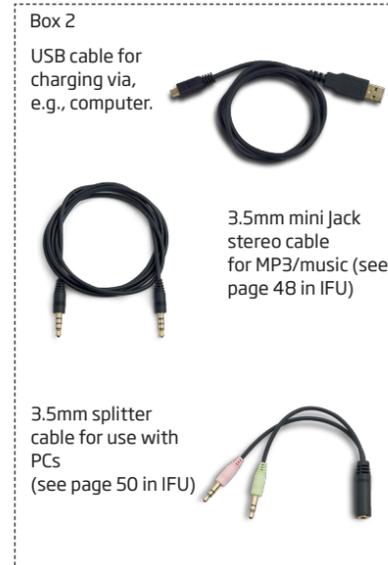

Box 2

USB cable for charging via, e.g., computer.

3.5mm mini Jack stereo cable for MP3/music (see page 48 in IFU)

3.5mm splitter cable for use with PCs (see page 50 in IFU)

Introduction


Thank you for choosing our product as your means to better hearing.

Streamer Pro offers a wide range of connectivity solutions tailored to improve your everyday life with Oticon hearing instruments.

For detailed information, secondary functions, or more advanced functions, please refer to the included Instructions For Use or materials, including instructional videos, online:

www.oticonusa.com/connectline

Wearing Streamer Pro

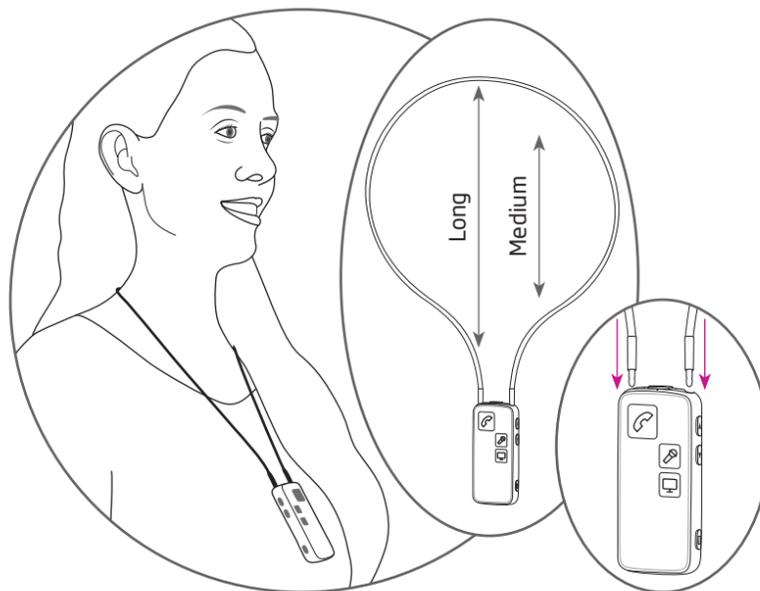


The neck loop is an integral part of the Streamer's electronic system. It is used as an antenna to transmit sound to your hearing instruments.

For best possible sound and longest possible streaming time, the Streamer should always be worn with one of the included neck loops, as in the above figure.

More information: page 16 in the Instruction For Use.

Charging Streamer Pro

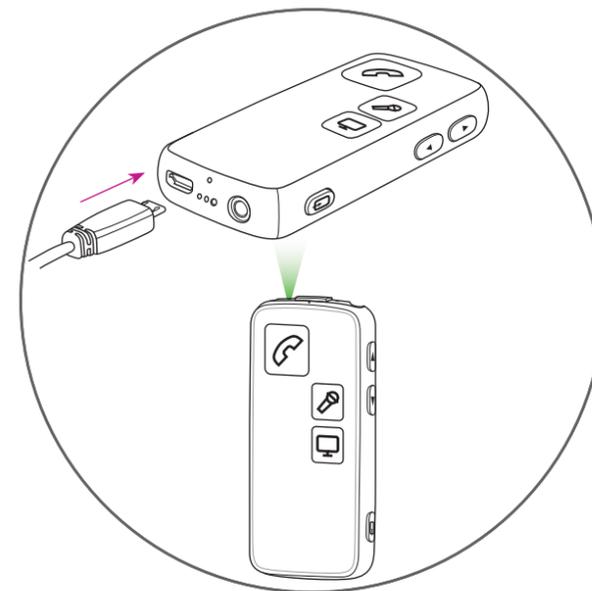

Insert cable from the enclosed charger in the micro USB socket on the bottom of the Streamer.


Charging is indicated by the blinking green on/off indicator on the top.

We recommend that you charge the Streamer daily, e.g., every night.

More information: page 22 in the Instructions For Use.

Power on

Press the on/off button for approx. 2 seconds until all front lights are on, and the on/off indicator on the top is green.

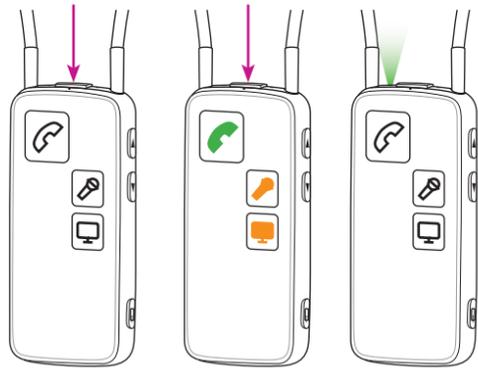

Power off

Press and hold the on/off button for approx. 2 seconds until the green on/off indicator is turned off.


More information: page 20 in the Instructions For Use

Pairing the Streamer

Before the Streamer can be used with other devices - such as cell phones - the two devices have to be paired.

Bring the Streamer into pairing mode. Press and hold the on/off button for approx. 5 seconds, until the on/off indicator blinks blue. The on/off indicator will switch to steady blue to indicate successful pairing.

Both the Streamer and the cell phone need to be in pairing mode. The pin code for the Streamer (if requested) is 0000 (four zeros).

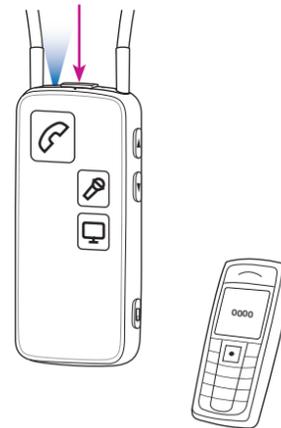


More information: page 30 in the Instructions For Use

Cell phone

Answer a call

When your cell phone rings, you will hear an incoming call alert in your hearing instruments, and the phone button on the Streamer and the status indicator will blink green.

Press the phone button briefly to answer the call.

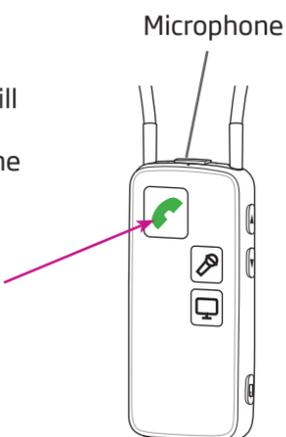

Reject call

An incoming call can be rejected by briefly pressing the volume down button.



Make a call

To make a call, you only need to operate the cell phone. The call is automatically connected to your Streamer.



More information: page 35 in the Instructions For Use

Remote control of the hearing instruments


The Streamer functions as a remote control for your hearing instruments when not used for streaming.

Volume up/down

Short press up or down

Change program in the hearing instruments

Press & hold up or down

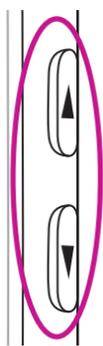


More information: page 28 in the Instructions For Use

Music

You can listen to music via the mini jack cable (plug-in on bottom of the Streamer) or wirelessly via Bluetooth.

Music is chosen by a short press on the AUX selector.

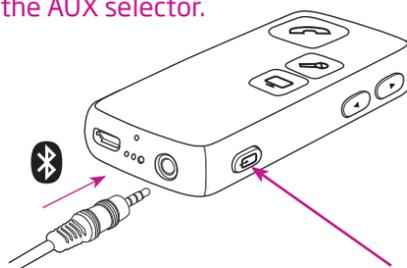


More information: pages 48 and 52 in the Instructions For Use

T-coil

The Streamer has a built-in tele loop receiver (T-coil).

To receive sound from the tele loop receiver press and hold the AUX selector for approx. 2 seconds.

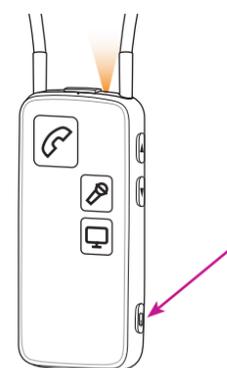

More information: page 56 in the Instructions For Use

ConnectLine Phone

Note: requires additional equipment.

Makes it possible to use the Streamer with a traditional analog landline phone.

Press the phone button briefly to take a call.

More information:
www.oticonusa.com/connectline



ConnectLine TV

Note: requires additional equipment.

Makes it possible to transmit the sound from a TV to your hearing instruments through the Streamer.

Press the TV button briefly to activate the TV audio stream.

More information:
www.oticonusa.com/connectline



ConnectLine Microphone

Note: requires additional equipment.

Makes it possible to transmit the sound from a chosen speaker to your hearing instruments through the Streamer.

Press the microphone button briefly to activate the microphone.

More information:
www.oticonusa.com/connectline

