

People First

People First is our promise to empower people to communicate freely, interact naturally and participate actively

INSTRUCTIONS FOR USE

MICROPHONE STREAMER PRO

1449301US / 04.14



What's in the box?

Microphone



Carry pouch



Lanyard



Power supply (charger)



Important Information





This manual




Product overview

- Status indicator (orange)
- Microphone input (Voice)
- Power indicator (green)
- On/off button
- Microphone input (noise detector), do not cover

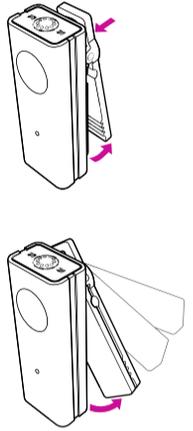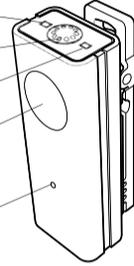

How the Microphone works

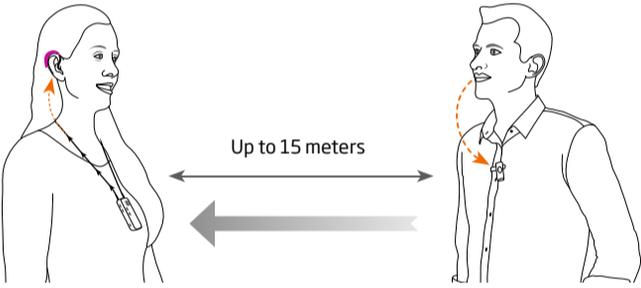


Hearing Instrument user


Speaker

The Microphone will transmit the speech from the Speaker directly to the Hearing instrument user.
Note: A Microphone will only work with one Streamer at a time


Wearing with Clip

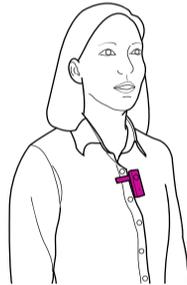

Wearing with Lanyard


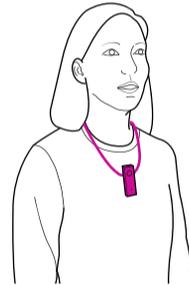


Incorrect usage

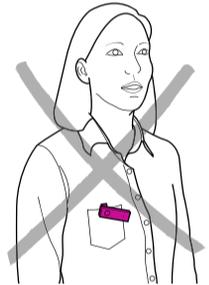

The microphone should point towards the speaker's mouth and be within a distance of 10 to 20 cm.

Charging / power indicator

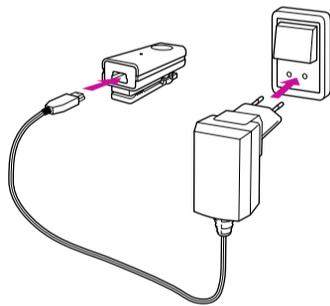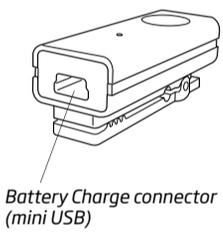

Constant red light



Blinking green light



Constant green light



To charge the microphone, connect the power supply cord to the mini USB connector at the bottom of the microphone.

Battery Low
Maximum 20 min. speaking time left

Charging

Fully charged
(After approx. 1.5 hours.) Will last for approx. 5 hours use.

Pairing / initial setup (only needed once)

1. Streamer Pro in pairing mode



Press

6 secs



Hold until blue light flashes

2. Microphone in pairing mode



Press and hold for 6 seconds until orange Light flashes rapidly

Orange light is flashing rapidly in pairing mode



6 secs

3. Microphone paired



1 min

Wait up to 1 minute until Streamer light is orange and microphone light is orange

People First

People First is our promise to empower people to communicate freely, interact naturally and participate actively


INSTRUCTIONS FOR USE

MICROPHONE STREAMER PRO



Start / turn on / resume sound from Microphone

1. Make sure Streamer Pro is on

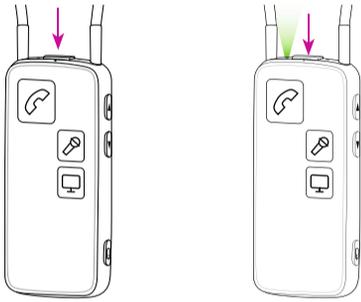

Press the on/off button on the top

until all lights on the front and top flash to indicate it is powered on

2. Start / turn on Microphone



Short press on on/off button



Orange light slowly pulsating

3. Sound from Microphone to Streamer Pro



Orange light on Streamer is on and orange light on Microphone is on

Pause sound


a. Pause sound from Microphone



Short press on on/off button




Orange light on Microphone turns off

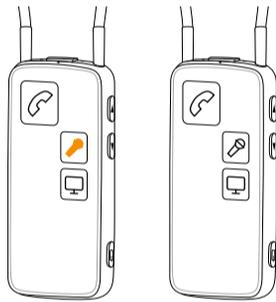

Orange light on Streamer turns off

b. Pause sound from Streamer Pro

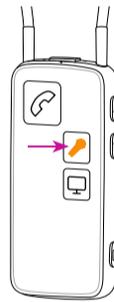

Short press on Streamer Mic button



Orange light on Streamer off



Orange light on Microphone off


Resume sound from Streamer Pro

1. Microphone has to be on



(green light on)

2. Start sound from Streamer Pro

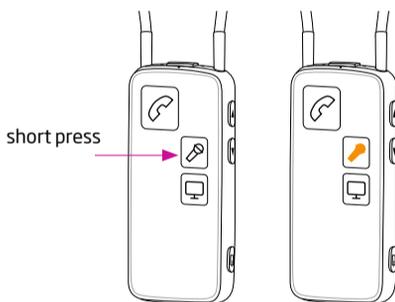

short press

Short press on Mic button



Orange light on Streamer slowly pulsating

3. Sound from Microphone to Streamer Pro



After 2 seconds, orange light on Microphone turns on

Check sound



Tap the top of the microphone to check that the sound is heard in the hearing instruments

Turn off Microphone



Press and hold two seconds, until green light turns off