

# Oticon ON guide


Create an Oticon ON account and set-up the Oticon ON app to use IFTTT

## What you will need:

Open hearing aid · iPhone 5 (or later) · Oticon ON app

1

Open the Oticon ON app on your iPhone.

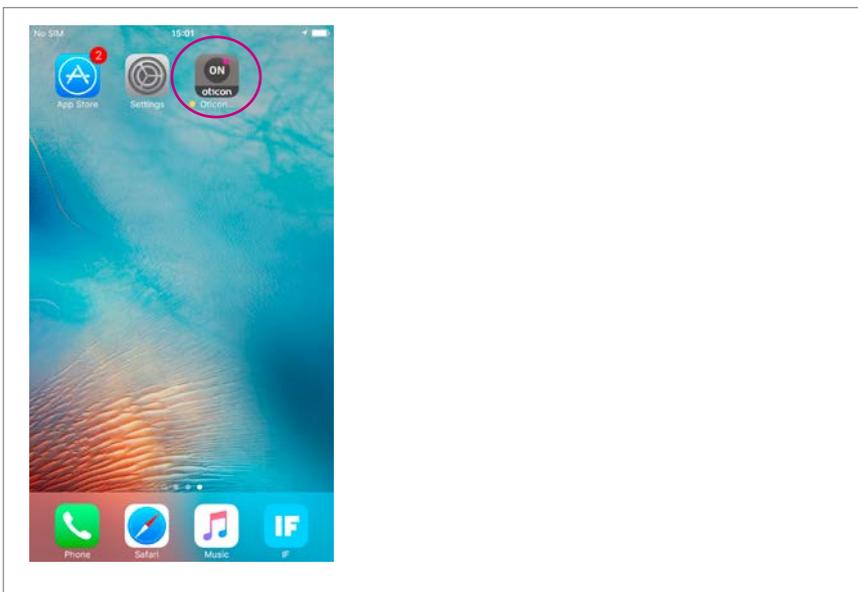

**Note:** If you are launching the Oticon ON app for the first time, you will be asked to share your location. Press 'Allow' if you want to enable 'Find my hearing aids'.


Allow "Oticon ON" to access your location even when you are not using the app?

The app uses location services to help you find your hearing aids.

Don't Allow

Allow

Follow the connection guide and read through the ON app instructions.

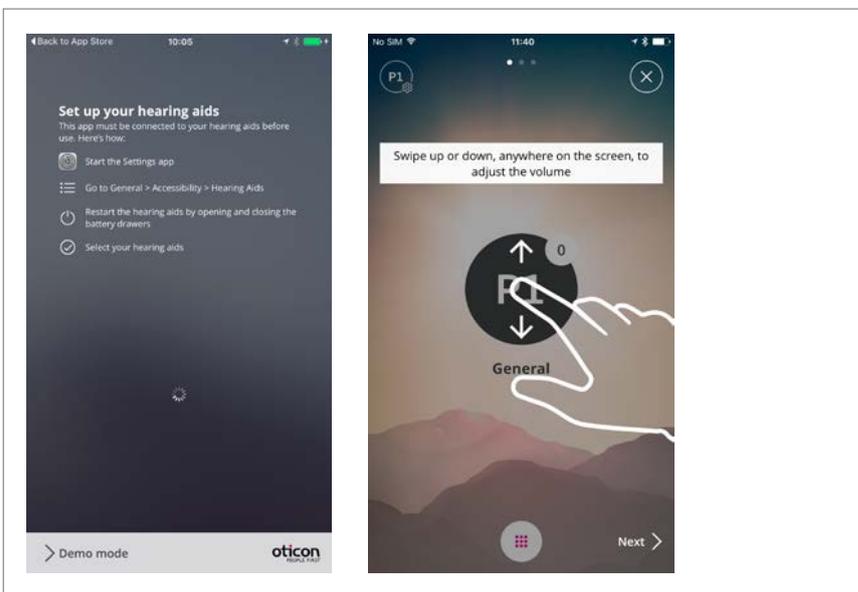


2

Press on the menu button and then click on the 'IF' button.



3

Press 'Get started'.

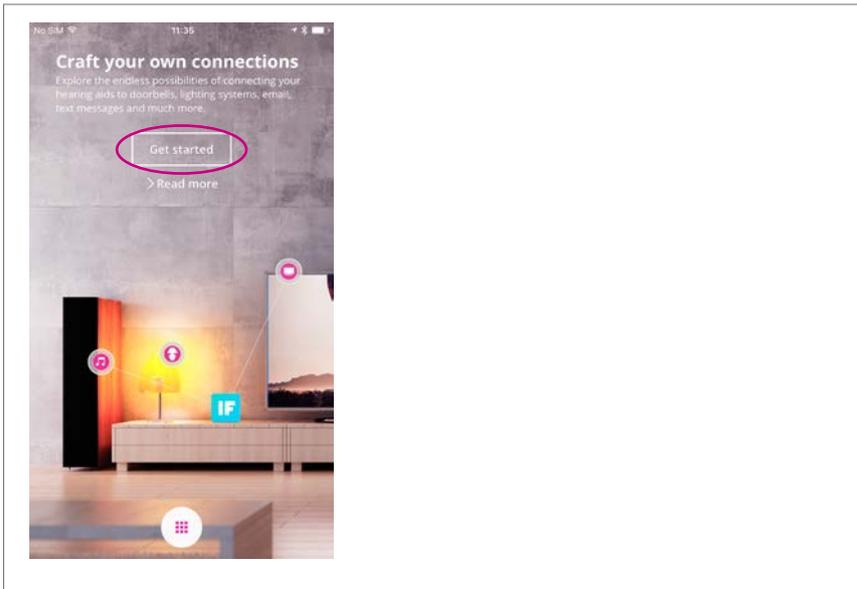

4

If you already have an Oticon On account use it to sign in.  
If you don't have an Oticon On account, you can create one using your Facebook or Google account (see step 4a) or with your email address (see step 4b).

4a

#### Sign-up using Facebook or Google

Click on either 'Facebook' or 'Google' and log-in using your account information. Then click on 'accept'.

The screenshot shows the Oticon sign-in page. At the top left is a 'Cancel' button. The Oticon logo 'oticon PEOPLE FIRST' is centered. Below the logo, there are two sections: 'Sign in with your social account' and 'Sign in with your existing account'. The social account section has buttons for Facebook and Google, which are circled in pink. Below this is an 'OR' separator. The existing account section has input fields for 'Email Address' and 'Password', and a 'Sign in' button. At the bottom, there is a link for 'Don't have an account? Sign up now' and a link for 'Forgotten password'.

4b

#### Sign-up using email

Click on 'Sign up now'.  
Enter your email address and then click on 'Send verification code'.

The screenshot shows the Oticon sign-up page. At the top left is a 'Cancel' button. The Oticon logo 'oticon PEOPLE FIRST' is centered. Below the logo, there are two sections: 'Sign in with your social account' and 'Sign in with your existing account'. The social account section has buttons for Facebook and Google. Below this is an 'OR' separator. The existing account section has input fields for 'Email Address' and 'Password', and a 'Sign in' button. Below the existing account section is a 'Sign up now' button, which is circled in pink. To the right of the sign-up section, there are input fields for 'Email Address', 'New Password', 'Confirm New Password', and 'Display Name', and a 'Send verification code' button. At the bottom, there is a link for 'Don't have an account? Sign up now' and a link for 'Forgotten password'.

Go to your email inbox and open the verification email that has been sent to you.  
Copy the verification code from the email, paste it in to the 'Verification code' field, and click on 'Verify code'.

The screenshot shows an email verification page. On the left, there is a blue header 'Verify your email address'. Below it, the text says 'Thanks for verifying your - your email - account!' and 'Your code is: 147450'. Below that, it says 'Yours sincerely, Oticon'. On the right, there is a sign-up form with input fields for 'Email Address', 'Verification code', and 'Display Name'. The 'Verification code' field is filled with '147450'. Below the form are buttons for 'Verify code' and 'Send new code'.

Enter and confirm a new password for your account. Choose a 'Display name' and click on 'Create'.

The screenshot shows the Oticon account creation page. At the top left is a 'Cancel' button. Below it are input fields for 'New Password' and 'Confirm New Password'. Below these is an input field for 'Display Name'. At the bottom are buttons for 'Create' and 'Cancel'.

5

Confirm that IFTTT service and notifications are enabled. If not enabled, click on the 'IFTTT' and 'Notifications' sliders so they are green.

