

For more detailed instructions please see the Installation Guide enclosed with your system.



### Power Outlet

#### Box 1

TV Adaptor Power Cord



**STEP 1:** Plug the power adaptor into the power source.



**STEP 2:** Plug the power cord into the power adaptor marked A.



**STEP 3:** The light will be green when power is on.

### TV Cord

#### Box 2



TV cord



**STEP 1:** Locate audio output jacks on back of TV (typically labelled Audio Out).



**STEP 2:** Plug the TV cord into the output jacks as shown.



**STEP 3:** Plug TV cord into adaptor marked B.

### Streamer & Pairing

Streamer



**NOTE:** Before pairing, charge Streamer or attach to power supply.

Pairing button



**STEP 1:** Place Streamer close to adaptor. Press pairing button until blue light flashes rapidly.



**STEP 2:** Pairing is complete when orange status light on TV adaptor comes on (about 1-2 minutes).



**STEP 3:** The TV adaptor can be mounted to the TV with the Velcro provided.



#### Box 3

#### Alternative Connections:

Use the Audio Jack Adaptor or Microphone (Box 3) if the audio output jack is not available. See enclosed booklet or installation instructions.

Audio Headphone Jack



TV Microphone



For more detailed instructions please see the Installation Guide enclosed with your system.



### Power Outlet

#### Box 1

Phone Adaptor Power Cord



**STEP 1:** Plug power adaptor into power source.



**STEP 2:** Plug the power cord into the power adaptor as shown.



**STEP 3:** The light will blink red when power is on.

### Phone Cord Connection

#### Box 2



**STEP 1:** Unplug phone line from the phone or cordless phone base station.




**STEP 2:** Plug the phone line into Phone adaptor marked LINE (black socket).



**STEP 3:** Other end of cord remains in the wall jack.



**STEP 4:** Plug the Tele Cord into the Phone Adaptor marked PHONE (yellow socket).

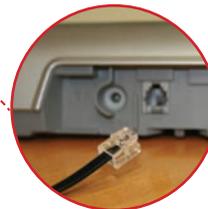

**STEP 5:** Plug the other end of the Tele Cord into the phone or base station.



Tele Cord

### Streamer & Pairing


Streamer



Pairing button



**STEP 1:** Place Streamer close to adaptor. Press pairing button until blue light flashes rapidly.

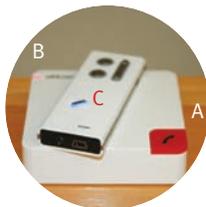

**STEP 2:** Press red button (A) on Phone Adaptor until red status indicator is flashing (B).

After 5-10 sec both the red light (B) and the blue (C) will be constant. Blue light will then fade to off (C).



**STEP 3:** Verify pairing by pressing the phone button (A) on streamer. The red status button will flash (B).

