

# Using TV Adapter 3.0

1

## Connect the TV Adapter 3.0 box to your television


See the Instructions for Use booklet that comes with the TV adapter. Four methods are available to support a wide variety of connection needs.

2

## Pair your hearing aids with the TV Adapter

Before using the TV adapter, it must be paired with your hearing aids. Pairing is only required once and takes around 20-60 seconds.

1. Make sure the TV adapter is on (plugged in).
2. Open and close the battery door on (both of) your hearing aid(s).
3. Place both hearing aids close to, or on top of, the TV Adapter.
4. Pairing is completed when the Power Indicator and TV Indicator lights turn steady blue for about 30 seconds.
5. Your hearing aids are now able to receive sound streaming from the TV adapter.

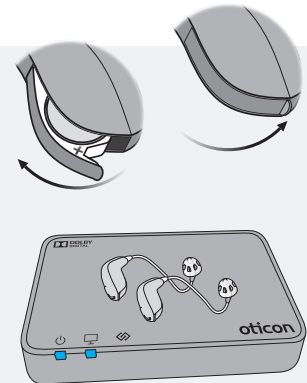


3

## Start TV sound using your hearing aids


To listen to TV, simply make long presses on the button on your hearing aids to cycle through the programs until you reach the TV listening program. The program cycle will depend on how many programs have already been set up in your hearing aids by your hearing care professional.

To stop the sound from the TV, simply cycle back to the desired program in your hearing aids.

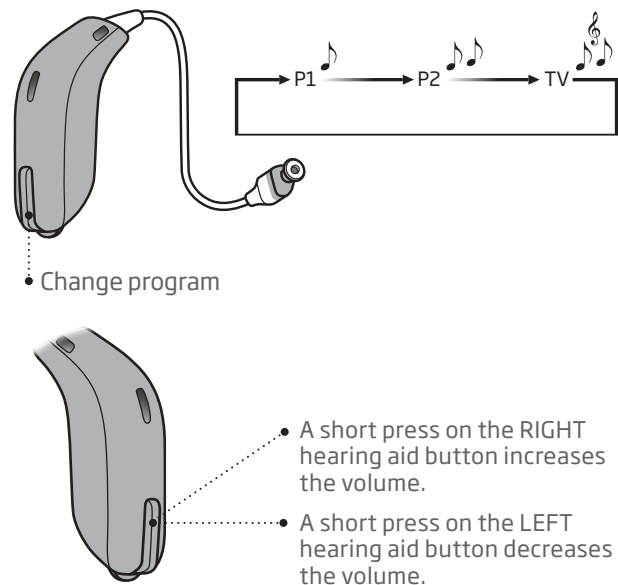

4

Alternatively, you can also start TV sound and adjust volume with the optional hearing aid Remote Control 3.0 accessory.

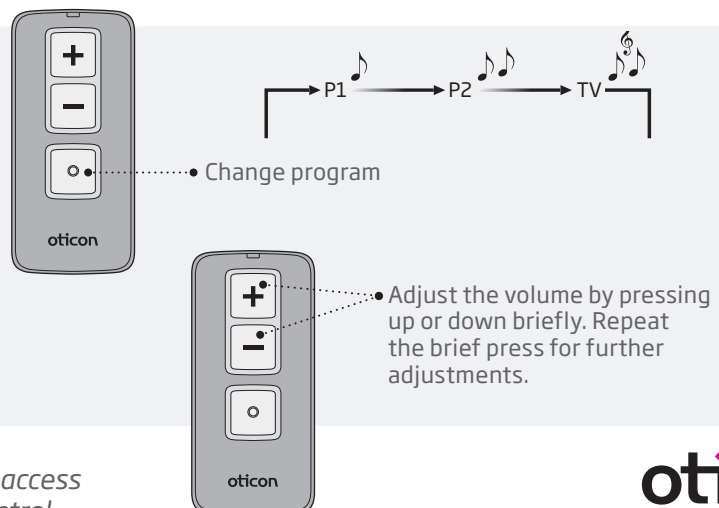


**Note:** The optional Oticon ON App also lets you access the TV listening program and adjust volume control.

# Using Remote Control 3.0


- 1 Before the remote control can be used with your hearing aid(s), they need to be paired with your hearing aid(s).

Check that the remote control has working batteries by pressing any button. (The status light at the top should light up if the batteries are good.)


- 2 Open and close the battery drawer on both of your hearing aids.

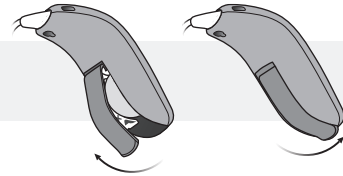


- 3 Place both of your hearing aids within range of the remote control. (within 20 cm, approximately 7 inches)

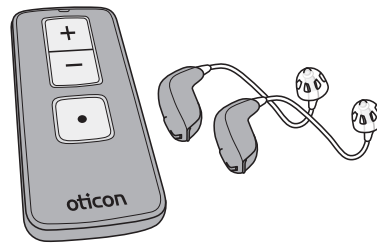

- 4 Press any button on the remote control and it will automatically pair with the hearing aids in range.

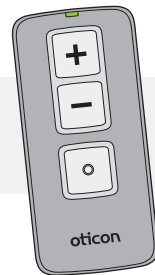


- 5 When the status light on the remote control lights up steady green for 10 seconds, the devices have been paired successfully.

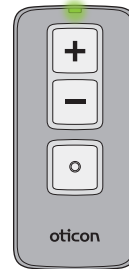


## Operating Range

You can operate your hearing aids with the remote control from a distance of up to 1.8 meters. (approximately 6 feet)


- 6 You can change hearing aid programs and volume with the Remote Control.

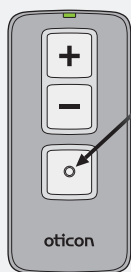

Press to change program

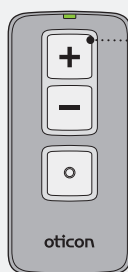

Press "+" to increase volume (louder sound)

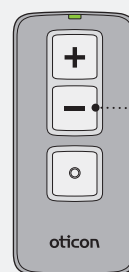


Press "-" to decrease volume (softer sound)

## Mute

Use the mute function if you need to silence the hearing aids while wearing them. Press and hold the volume down button for approximately 2 seconds until the status light turns off, to mute the hearing aids.

## Unmute

Briefly press any button on the remote control to unmute your hearing aids.

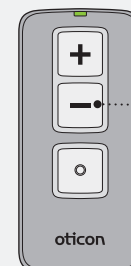

Press for approximately 2 seconds